

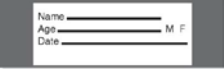

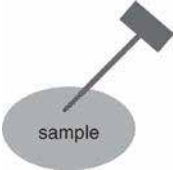


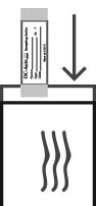
Taropwe Epwe Aituk Usun Aean Ei Kit

Aneani meinisin taropwen usun aean ei mettoch memwen kopwe fori. Pwungun ei test epwe tongeni mwan ika kose fori pwungun angangan.

Watiw ewe Pwuse

1. Wanong na taropwe mei kawor non toilet bowl won na konikin na toilet.
2. Pinchotiw won na taropwe.
3. Mina na pwuse ngeni na fochun mettoch e kawor memwen na taropwe epwe kokotiw me na pwuse epwe pachengeni na konik.
4. Flushuni na toilet. Na taropwe a tongeni flush ese tongeni osukosuka na septic pwun mei biodegradable.

Angei ewe Pwuse

1. Amasowa na taropwe e nomw won na pen nenien na pwuse ren omw information.
 
 Suki na opwon mei araw ina kopwe okunu me weitata
 
2. Aroti nukun na pwuse ngeni na fochun mettoch mei kawor.
 
 Ikanaia mei wor usun mwaran garang won na fochun epwe pwonuno ren pwuse.
 
3. Wanong na fochun non na nenian iwe ka opungano ngeni na opwonun ongutano epwe ngut. Kosapw pwan suk sefani.
 
 Eniwini na pwuse ke fori ngeni noumw ewe doctor ika ewe laboratory non na envelope e kawor. Kose mwochen kosapw mailini ngeni Polymedco.
 

Please include CLH requisition with your specimen in the return envelope.

Please visit our website to access a collection instruction video:
<http://www.clinicallabs.com/physicians/collection-instructions.aspx#FOBT>

Client Services